

CALF SCOUR TEST KIT

An ideal aid in the rapid detection of rotavirus, coronavirus, E. coli and cryptosporidium parvum in calves.

PRODUCT BENEFITS:


- No need to wait for lab results, just test and go in 10 minutes!
- Detects E. coli F5 (ETEC) (K99), rotavirus type A, coronavirus (BCoV) and cryptosporidium parvum.
- Each kit contains 5 calf tests.
- Simple visual confirmation.
- Highly sensitive and reliable.



INSTRUCTIONS FOR USE:

1. Take the faeces directly from the rectum of the calf. If the samples are liquid, take a spoonful.
2. If faeces are solid, remove the excess amount using a spatula.
3. Dilute the spoon content in the liquid of the small tube called sample tube.
4. Shake the sample tube to homogenize.
5. Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.
6. Remove the tube's protective caps.
7. Break the blue end.
8. Place the cartridge on a completely level surface. Deposit three drops in each small hole. Leave the device on a flat surface and wait for 10 minutes. The liquid contained in the sample tube moves to the strip and slowly migrates along the strips.

9. After 10 minutes, read the results using the picture below as standard.

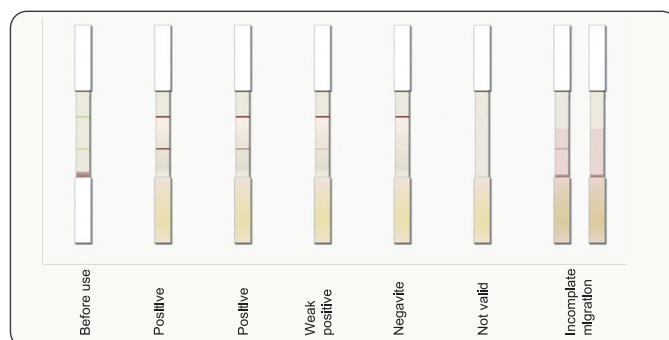

10. If test is positive, seek veterinary advice immediately.

SHELF LIFE:

2 years from date of manufacture.

STORAGE:

Store between 0°C to 25°C.

Keep out of direct sunlight.

For health and safety information, please refer to the Safety Data Sheet.

SIZES AVAILABLE:

- P829 – 1 kit (5 calf tests)